Digiloop 50

- NL Handleiding ringleidingversterker
- EN User guide loop amplifier
- DE Gebrauchsanleitung Ringschleiferverstärker
- FR Manuel du amplificateur de boucle d'induction magnetique

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Introduction

Congratulations on purchasing the Digiloop 50 loop amplifier.

The Digiloop 50 has been especially designed for hearing aid users in order to reproduce the sound of television or audio devices without ambient background noise. The Digiloop 50 uses the latest digital technology. This means the sound is reproduced without distortion, the generation of heat is minimal and the Digiloop 50 is more economical in the use of energy.

The integrated switch unit makes it easy to choose from various sound sources without needing to swap over wires.


Operation of loop amplifiers

A loop amplifier is designed to make the sound of television or audio installations audible by a hearing aid without interference from external sound sources.

The loop amplifier (Digiloop 50) is connected to the television and/or audio installation and the sound of the television or audio installation is passed to the loop amplifier. Connected to the loop amplifier is a loop called the loop cable, in which a magnetic field is generated. By setting the hearing aid (or hearing aids) to T (telecoil) or MT (combined microphone-telecoil) the sound from the television or audio-installation is made audible for the hearing aid user.


The loop cable is installed in the room where a loop amplifier is required.

Description of Digiloop 50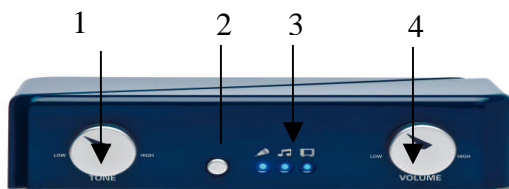

- 1 tone control
- 2 selector switch
- 3 LED indication
- 4 volume control

Figure 1, Front view

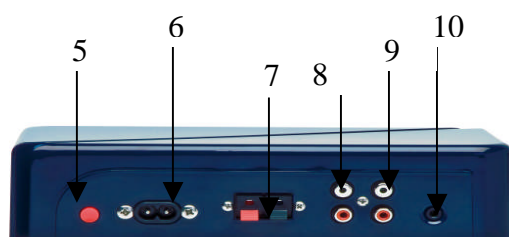- 5 on/off switch
- 6 mains connection (230V)
- 7 loop cable connection
- 8 TV
- 9 audio
- 10 microphone

Figure 2, Rear view

Assembly


Loop cable

Place the Digiloop 50 in the vicinity of a sound source (e.g. TV, audio installation). Form a loop with the loop cable (Figure 3).

Lay one end of the loop cable next to the Digiloop 50 and lay the loop cable around the room where it is to be used, ensuring that the other end reaches the Digiloop 50. Attach the stripped ends to the loop amplifiers connection (Figure 2, No 7). The cable is normally laid on the floor. Ensure that there are no kinks in the cable. The cable can be attached to the skirting board or laid beneath the floorboards. The cable can be placed in front of a door sill.

Note: if there are no door sill(s) and you run the loop cable up over the door frame the magnetic field in the room may not be the optimum field possible.

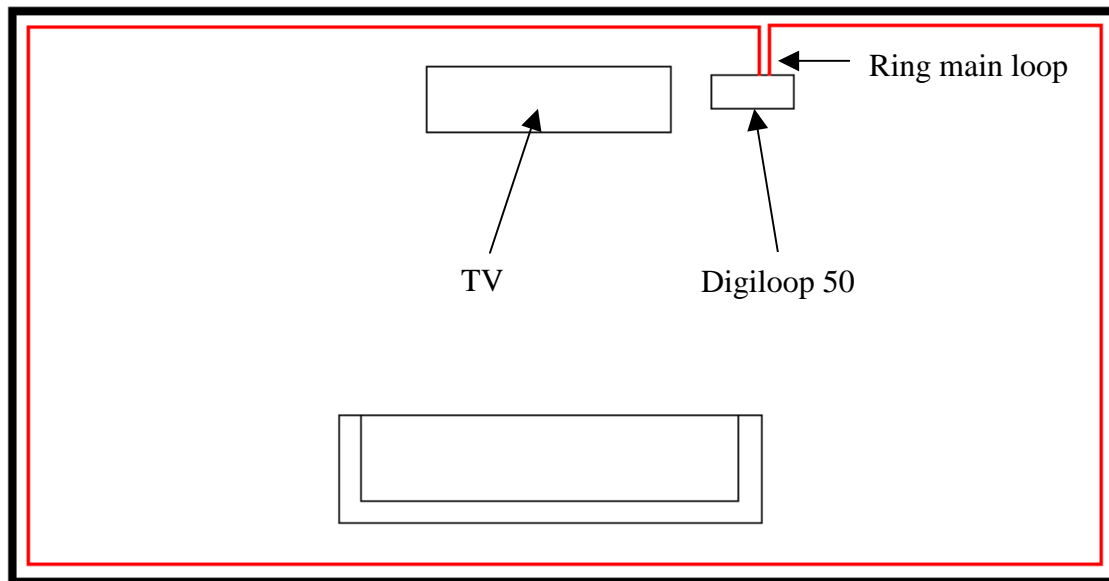Figure 3

Audio connections

The Digiloop 50 is fitted with three audio connections;
2 x line inputs for TV () and audio () and 1 x microphone input ().

TV

If your TV has a free scart, you need a scart plug. Connect this to one of the audio leads. Connect the other end of the audio lead to the TV input of the Digiloop 50 (Figure 2, number 8).

If your TV has an audio output (RCA, cinch) you can connect your audio lead directly to it and establish a link to the TV input of the Digiloop.

Audio

The output of your audio installation is connected via the audio lead to the input of the digiloop. (Figure 2, number 9)

Microphone

A microphone (not supplied) can be connected to the Digiloop 50 using a 3.5 mm jack plug. (Figure 2, number 10)

Operation

Selector switch

The selector switch allows you to choose from various connected audio sources. By pressing in the knob (Figure 1, number 2) you can select a different output. Your choice is clearly displayed by the illuminated blue LED (Figure 1, number 3).

Volume control

The volume of the Digiloop 50 can be adjusted using the right hand rotary knob (Figure 1, number 4).

To increase the volume, turn the knob clockwise.

To reduce the volume, turn the knob anti-clockwise.

Tone control

The tone colour of the sound can be changed by using the left-hand rotary knob (Figure 1, number 1).

To increase the tone, turn the knob clockwise.

To reduce the tone, turn the knob anti-clockwise.

Problems and solutions

Problem	Causes and solutions
No sound	<ul style="list-style-type: none"> - Check the mains voltage connection - Check that the audio leads are correctly connected - Check that the loop cable is correctly connected. - Check that one of the blue LED's is illuminated. - The amplifier is not switched on, press down the red mains switch on the back of the unit. - Check that the selection switch is set to the desired sound source. - Check that your hearing aid is set to (telecoil) loop amplifier (T).
Blue LED not illuminated	<ul style="list-style-type: none"> - Check mains voltage cable. - Check that the amplifier is switched on (the red mains switch on the back must be pressed in)
The sound is distorted	<ul style="list-style-type: none"> - The input signal is too loud, switch the sound source to another input channel of the Digiloop 50. - If you are using a microphone the microphone (input) may be faulty.
The sound is too soft	<ul style="list-style-type: none"> - Increase the volume of your hearing aid slightly - Increase the volume of the digiloop slightly
Hum/buzz sound	<ul style="list-style-type: none"> - Switch the Digiloop 50 off and keep your hearing aid set to T. If the hum is still present, the interference could be caused by lighting (dimmers), TV set or induction cooking hobs. Switch off the source of interference.

Safety instructions

The manufacturer cannot be held responsible for damage caused by operation that does not conform to safety instructions.

- Do not open the housing of the Digiloop 50.
- Clean only with a dry cloth.
- Do not place the loop amplifier near heat sources such as radiators, fans, cookers or other apparatus that produce heat.
- Ensure that the cables cannot be trodden on and that the cables are not trapped.
- Remove the plug of the loop amplifier from the socket during storms or if you are not going to use the equipment for an extended period.
- Check that the mains plug is not in the socket when connecting the loop amplifier.
- Ensure all maintenance is carried out by qualified personnel. Maintenance is necessary if the loop amplifier is damaged, if it has got wet, does not work normally or if the amplifier has been dropped.

Technical specifications:

Dimensions:	length 22.5 cm; width 12 cm; height 5 cm
Weight:	827 gram
Mains lead input:	C8
Supply voltage:	230 V a.c.
Power consumption:	Max. 20 VA
On/Off switch:	Yes (primary side transformer)
Operational indication:	Yes, with LED from input selection.
Tone control:	Single knob, high/low +/- 6 dB
Volume control:	Yes
Output:	Current controlled
Output current:	1 A continuous, 4 A peak
Loop:	Simple
Recommended loop resistance:	0.4.... 1 Ohm
Recommended wire diameter:	0.5 mm ² for lengths up to 20m 0.75 mm ² for lengths up to 30m 1 mm ² for lengths up to 40m
Surface loop:	Maximum 50 m ² (1mm ² wire)
Frequency range:	50 Hz - 8k Hz +/- 2 dB
Inputs:	2 x stereo RCA (cinch-cinch) 1 x microphone 3.5mm
Output:	Quick fitting connector (loudspeaker output)
Selector switch:	Yes, with LED indication
Warranty:	24 months, after purchase date

The product fulfils the requirements of the following EC directives:

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